

Reprocessing Flexible Endoscopes

Processing Endoscopes without LIC processor.

Wipe and leak test must be performed

Fill sink or basin with soapy water.

Place scope into basin.

Draw soapy water through and into lumens and let soak.

Wash external surfaces as well as brush and scrub internal lumens.

Rinse scopes external and internal surfaces with clean water.

Place scope into washer/disinfector.

Place correct connectors on the endoscope lumens.

When cycle is complete remove attached connectors.

If washer/disinfector does not have sanitized water rinse flush endoscope with 75% alcohol solution.

Processing Endoscopes WITH LIC processor.

Wipe and leak test must be performed

Place soiled endoscope into chamber, close Lid and press start button.

Remove validated CFU free endoscope.

Processing Endoscopes WITH LIC Cleaner.

Wipe and leak test must be performed

Place soiled endoscope into chamber, close Lid and press start button.

Remove validated cleaned endoscope from LIC Cleaner.

Place endoscope into washer/disinfector

Reprocessing Other Instruments

This is for instruments other than flexible endoscopes

